

41.

## Yeung Bone Harvest Kit. MK-BH26

The Bone Harvest Kit ensures the dental surgeon's easy and simple bone harvesting during surgery.



Revolutionary dual action-bone harvesting while drilling; The harvested bone can now be used elsewhere, as needed. A simple procedure enables the use of the patient's own bone, as opposed to substitute materials. Drills can also be used as final drills.

### Simple.

The bone will accumulate inside the drill's slot during drilling.

### Easy.

The Bone Harvest Kit can be used as a final drill while harvesting bone.

### Safe.

The Bone Harvest Kit can be used with MIS drill stoppers, ensuring a safe drilling procedure.

### Kit Contents

The Bone Harvest Kit includes these drills:



### Recommendation:

The drill can be used with external irrigation. Start drilling with a slight push and a little continuous pressure. When the drill is taken out, continue rotation to keep the bone inside the drill's slot.

\* The drills should be used for a maximum of 30 drillings.

### Recommended Speed:

Recommended speed is between 20-50 RPM without irrigation.

### Material:

The bone harvest drills are made of stainless steel and can be easily cleaned.

### Color coding:

 Drill diameter 2.80mm

 Drill diameter 3.20mm

 Drill diameter 3.80mm

 Drill diameter 4.50mm